

INSTRUCTIONS FOR USE
IFAK
Individual First Aid Kit

INTENDED USE: Individual First-Aid Kit (IFAK) is a procedure pack which contain devices intended for the first aid support to the individual.

INTENDED USERS: First responders and other subjects in need for the first aid support.

PERFORMANCE: It achieves its intended use by applying different pack devices according the need of the patient.

INSTRUCTIONS:

- 1.) Remove IFAK from packaging
- 2.) Attach IFAK to preferred area of uniform / gear / belt. Drop leg style units are often attached to the wearer's belt and upper thigh. Use the Molle straps to allow the IFAK to be secured to bags and other areas of equipment. This is the user's preference.
- 3.) Verify the zipper / bag is sealed and closed when not in use & in transport.
- 4.) When IFAK items are needed, open zipper and expose items inside IFAK. Try to keep the items clean and free from debris before they are put into use.
- 5.) Select the items needed and pull free from IFAK interior compartments. It is recommended that the user become familiar with each IFAK component location prior to use.
- 6.) See individual items instructions for information on their recommended usage steps.

MAINTENANCE & CLEANING: After use, close IFAK using zipper and straps. Replace used items before next use.

CONTRAINDICATIONS: None

WARNINGS, PRECAUTIONS & LIMITATIONS: None

SAFE DISPOSAL: The product should be disposed in the dedicated area as recommended by the local authorities.

NOTICE: Please report to FareTec or to the competent authority of the Member State in which you use this product any serious incident that has occurred and you suspect is in relation to this device. If you suspect a potential defect or a product has not met your expectations, please report this to the Company and provide as much information as possible. Company phone number: +1-440-350-9510











FareTec Inc, 1610 West Jackson St, Painesville, OH 44077 USA, (p) 440-350-9510
(f) 440-350-9520, www.faretec.com



CMC Medical Devices & Drugs S.L., C/ Horacio Lengo n18, C.P 29006
Málaga-Spain



1610 W Jackson Street • Painesville, Ohio 44077 • 440-350-9510 • www.faretec.com

	Manufacturer		Authorized representative in the European Community
	Medical Device		Importer
	Consult instructions for use		Catalogue number
	Date of manufacture		Batch code