



1610 W Jackson Street • Painesville, Ohio 44077 • 440-350-9510 • www.faretec.com

INSTRUCTIONS FOR USE LIFEBOX50

INTENDED USE:

LIFEBOX50 is intended for the protection, transport and temporary isolation of temperature-sensitive content.

INTENDED USERS:

First responders and other subjects in need of transport and temporary isolation of temperature-sensitive content.

PERFORMANCE:

LIFEBOX50 can maintain its interior temperature above or below the ambient temperature depending on the user preference.

INSTRUCTIONS:

- Insert any commercial cold packs. Place them around the payload to distribute the source evenly. It may be recommended to prevent any medications from touching the cooling sources directly. In this case, place a barrier between them.
- Locate the temperature probe and place it near the center of the payload.
- Once all the components are inside the LIFEBOX50, make sure no items come between the panel edges as the lid closes. Close the latch handles and lock if necessary (not included).
- Monitor the payload temperature periodically throughout transport and usage. The digital thermometer is located on the front of the LIFEBOX50.

LIFEBOX50 will often maintain its temperature for 40-50 hrs. This will vary based on the ambient temperature, the payload and the cooling sources. Open the Lid only when necessary, in order to extend this duration. If a longer temperature control period is needed, insert an additional cooling source and monitor.

Each LIFEBOX50 is tested prior to leaving the factory however it is recommended that a trial be conducted by each agency in its typical work environment and payload. This will give the user an idea of the performance they can expect.

MAINTENANCE & CLEANING: Clean interior surface if soiled, using a lightly dampened cloth and a small amount of soap. The lid should be left open afterward to dry sufficiently. The outside can be cleaned using only warm, soapy water. Never use abrasive detergents, steel wool, abrasive sponges or chemical solvents. After cleaning, dry all parts carefully.

CONTRAINDICATIONS: None

WARNINGS, PRECAUTIONS & LIMITATIONS: None

1610 W Jackson Street • Painesville, Ohio 44077 • 440-350-9510 • www.faretec.com

SAFE DISPOSAL: The product should be disposed in the dedicated area as recommended by the local authorities.

NOTICE: Please report to FareTec or to the competent authority of the Member State in which you use this product any serious incident that has occurred and you suspect is in relation to this device. If you suspect a potential defect or a product has not met your expectations, please report this to the Company and provide as much information as possible. Company phone number: +1-440-350-9510



FareTec Inc, 1610 West Jackson St, Painesville, OH 44077 USA, (p) 440-350-9510
(f) 440-350-9520, www.faretec.com



CMC Medical Devices & Drugs S.L., C/ Horacio Lengo n18, C.P 29006
Málaga-Spain



	Manufacturer		Authorized representative in the European Community
	Medical Device		Importer
	Consult instructions for use		Catalogue number
	Date of manufacture		Batch code